

**NQ SMELL.**

# Perfume free odour remover

Contributing to the quality of life



# NO SMELL. Perfume free odour remover

## Contributing to the quality of life

### Benefits

- Perfume free
- Skin-friendly, isotonic
- User-friendly, easy to apply
- Environmental-friendly, non-toxic
- Textile-friendly
- Non-flammable, permitted on flights
- Available in small size, fits in pocket and purse

### Indications

- During treatment of wounds (chronic, diabetic, pressure, venous, atherosclerotic, cancer, etc.)
- Stoma, catheter and accessories care
- Fistula care
- Palliative care
- Body odour and other odours
- Malodorous textiles and clothes

### Instruction for Use

1. Spray NoSmell directly on the odour source or on a clean cloth. Use the cloth to clean e.g. skin.
2. Spray into the air if needed.
3. Remove the odour source by washing or drying.
4. Repeat the treatment daily and/or when necessary.



## Product Information

### • Container

Aluminium can, pressurized with Nitrogen. Aluminium is recyclable while the pressure ensures that there is no contamination of the contents or air penetration.

### • Mechanism

When used directly onto the skin or around wounds, the odour molecules are inactivated.

No Smell eliminates unwanted odours during and post-treatment.

### • Storage

Store at room temperature. It is recommended to warm the container up to 37°C prior to use (heat cabinet or warm water bath).

### • Application environment

The product is isotonic and has a low buffer capacity. It does not cause discomfort or irritation and it does not alter the pH around wounds or on the skin.



*Tilt maximum 30° when in use*

## Packing Information

### Art. No. 21050 –

No Smell. Perfume free odour remover  
50 gram  
12 cans per pack



### Art. No. 21250 –

No Smell. Perfume free odour remover  
250 gram  
12 cans per pack



Produced by:



Distributed by:



S2Medical AB

[www.s2m.se](http://www.s2m.se)

Sunnorpsgatan 5,

582 73 Linköping

Tel: +46 8-70 000 50

Mail: [hello@s2m.se](mailto:hello@s2m.se)

